

# Clean hands save lives!

## Hygienius ProWash

Preprogrammed wash cycle is designed to match the World Health Organization Guidelines "Clean hands protect against infection", resulting in highly efficient water usage.



The Hygienius ProWash is entirely self contained and needs no mains water service or drainage connections.

Preprogrammed hand wash cycle

The ProWash wash / rinse cycle is designed to match the World Health Organization Guidelines "Clean hands protect against infection".

To start the timed wash programme:-

- 1. Place the hands by the Greensensor and it will flash.
- 2. Water runs for 4 seconds to enable the wetting of hands.
- 3. The water stops for 20 seconds to allow for the application of liquid soap and performance of the 9 step hand wash routine.
- 4. Hot water runs for 10 seconds to enable the soap to be rinsed off the hands
- 5. The total wash time is approximately 35 seconds.

## The Hygienius ProWash features:-

- · Hands free operation under warm running water.
- The water always at the optimum temperature of 40° 43°C.
- Highly portable the Hygienius ProWash can be plugged in to any convenient electrical socket.
- Nothing to reset or adjust simple daily hygiene routine.
- Highly efficient use of water up to 40 ten second hot washes per filling.
- The unit comes complete with an easy to use TEALtainer which ensures that fresh and waste water are kept separate and hygienically contained.

### **Specification**

Overall height 990mmHeight to bowl 900mm

• Width 330mm

Weight (dry)Power supplymains

### Construction

- The body is fully moulded in blue fleck high impact polyethylene.
- The bowl is white acrylic capped ABS.
- Comes complete with moulded electrical plug.
- TEALtainer II securely retains waste water in a separate heavy duty LLDPE polythene 10 litre bag.

### Included with the ProWash

- TEALtainer II water container.
- Stainless steel paper towel holder.
- "C" bracket for TEAL soap dispenser.







"Consider it done"

22B Old Hope Road, Kingston 5, Jamaica W.I.

Find us on: WhatsApp at 1-876-440-4400.

Call us at: 1-876-440-4400 &

1-876-395-4436.

Email us at: dr.sinclair@live.com. Fax us at: 1-876-906-7423.