

# TEALtainer

This TEALtainer fresh and waste water container is for use with the MediWash portable hand wash unit.

## User information

- 1. Warning** - switch off your MediWash before handling the TEALtainer.
- 2.** The waste water is contained within a heavy duty polyethylene bag - when the fresh water is used up, the waste bag will be full.
- 3.** Disconnect the TEALtainer from the fresh water pick up pipe (white cap) and remove from unit.
- 4.** Drain the TEALtainer completely and dispose of all water to a sluice or foul water drain.
- 5.** Flush out the waste compartment (red cap) with fresh water. Make sure that the waste bag is empty.
- 6.** Unscrew the white cap and fill the fresh water bag from the cold tap to just below the filler cap. Refit the white cap.
- 7.** Return the TEALtainer to the rear of the MediWash and reconnect the fresh water delivery pipe, **making sure that the waste pipe is inserted into the red waste cap hole.**
- 8.** Switch on the power. The unit will automatically run a 10 second cold water purge.

**Do not** re-use waste water.

For cleaning, maintenance and operator instructions, please refer to the user instructions on the back of the top panel on the MediWash.

All Teal mobile hand wash unit information and maintenance documentation can be obtained from the user manuals page of the Teal website:-

<https://www.tealwash.com/user-manuals/>



**Teal support Monday-Friday 8-5 (Fri 1pm)  
Helpline 0121 770 0593**

Teal Patents Ltd, Unit 2 Waterloo Avenue  
Chelmsley Wood Industrial Estate  
Birmingham B37 6QQ  
Tel: +44(0) 121 770 0593  
Email: [enquiries@tealwash.com](mailto:enquiries@tealwash.com)  
[www.tealwash.com](http://www.tealwash.com)